

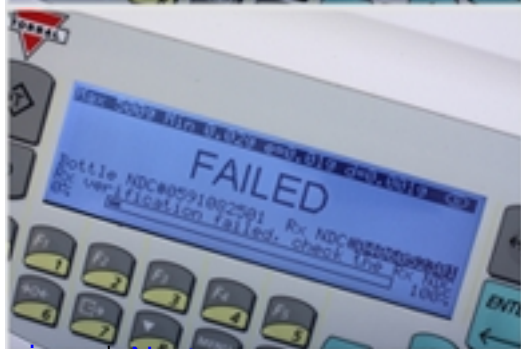


The NDC verification feature eliminates potential errors in prescription filling. It ensures that a prescription is correctly counted and filled. The feature allows the user to compare two NDC codes and check for a match. The NDC on your Rx label barcode is compared to the stock bottle barcode. If the two scanned NDC codes are the same the scale displays "Match OK" and allows to proceed with pill counting. If the codes are different, the scale displays a "Failed" message. Pill counting transactions which have failed Rx verification are recorded by the server in a dedicated report. As soon as "Failed" Rx verification occurs the server instantly sends out an email alert notifying the administrators. The NDC verification feature can be used as an integrated part of pill counting, or as a separate function to verify medication other than pills or tablets, such as creams, syrups and inhalers.

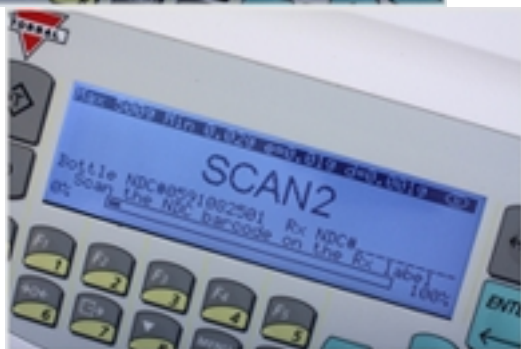

Rx NDC Verification

Wednesday, 27 April 2011 19:13 - Last Updated Wednesday, 27 April 2011 19:49

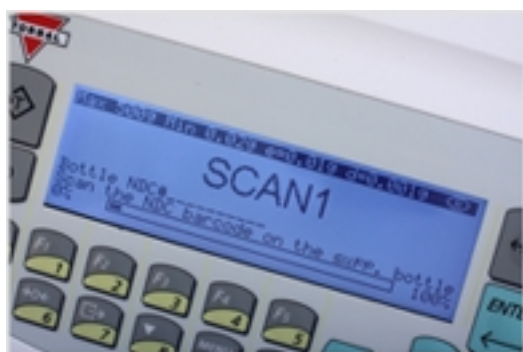

< [Previous](#) | [Next](#) >